In Vitro Diagnostic Use • Do Not Reuse

INSTRUCTIONS FOR USE

You must follow the test directions carefully to get an accurate result.

This test may not detect HIV infections that have occurred within 3 months of a possible exposure.

You are encouraged to read the booklet "HIV, Testing & Me" before running the test.

Do not eat, drink or use oral care products (such as mouthwash, toothpaste or whitening strips) 30 minutes before starting the test.

If you are on antiretroviral drugs (ARVs), including for HIV treatment, pre-exposure prophylaxis (PrEP), or post-exposure prophylaxis (PEP), you may get a false result.

VIEW INSTRUCTIONS
oraquick.com

HOW TO USE THE ORAQUICK™ HIV SELF-TEST KIT

YOU WILL NEED A WAY TO TIME THE TEST


Kit Contains: **Pouched device(s)** with a **test stick** and **test tube** (containing 1 mL of fluid), **test stand**, **Instructions for Use**, and **"HIV, Testing & Me" booklet**.

Your test kit may contain one or two pouched devices.

Tear open the pouch containing the tube.

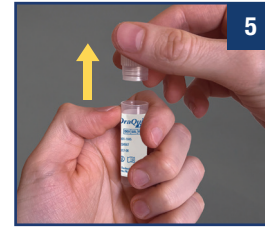Remove the cap.



DO NOT pour out the liquid. **DO NOT** drink.


Slide the tube into the **stand**.


Tear open pouch containing the **test stick** and remove. **DO NOT** touch the flat pad with your fingers. **DO NOT** eat or swallow the preservative.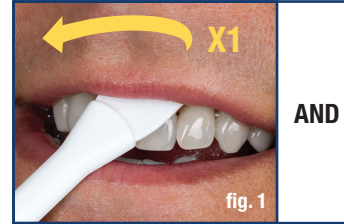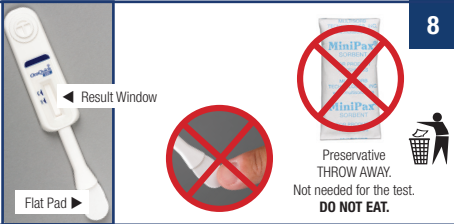

Press the **flat pad** firmly against your gum and swab it along your **upper gum once** (fig. 1) and your **lower gum once** (fig. 2). Make sure you swipe **each gum only once** or your results could be wrong.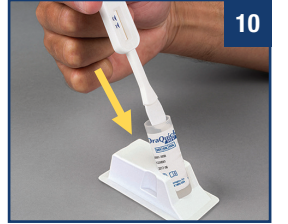

Put the **flat pad** all the way into the tube until it touches the bottom.

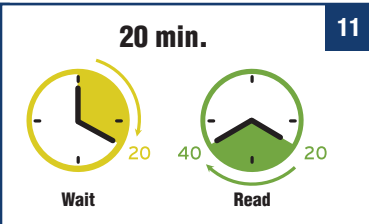LEAVE IT THERE for 20 MINUTES before reading the results. **DO NOT** read the result after 40 minutes.

INTERPRETING RESULTS Read test results in a well-lit area

HIV POSITIVE RESULT

Two complete lines, even if the line is faint, means you may be HIV POSITIVE and you need to seek additional testing by a trained professional to confirm an HIV diagnosis.

As soon as possible...

Visit your nearest **HIV Testing Center or Health Facility**

HIV NEGATIVE RESULT

IF READ BEFORE 20 MINUTES, RESULT MAY NOT BE CORRECT

ONE LINE next to the "C" and NO line next to the "T", your result is HIV NEGATIVE.

- If you may have been exposed to HIV, test again in 3 months.
- Seek regular testing if you engage in activities that increase your risk of HIV exposure.
- Consult the HIV, Testing & Me Booklet.

INVALID RESULT

If there is no line next to the "C" (even when there is a line next to the "T"), the test line or control line are not complete (all the way across the window), or a red background makes it impossible to read the test, the test is not working and should be repeated. **You will need to obtain another test.**

The test did not work properly.

Visit your nearest HIV Testing Center or Health Facility to test again.

NOT SURE OF RESULT

You do not know your result or you are unsure of your result. Visit your nearest HIV Testing Center or Health Facility to test again.

DISPOSE

Remove the test stick, put the cap on the test tube and throw away all contents in the normal trash. If you have more than one test device, **KEEP THE TEST STAND, INSTRUCTIONS FOR USE, AND "HIV, TESTING, & ME" BOOKLET PROVIDED TO COMPLETE ADDITIONAL TESTING.** Once all devices have been used for testing, throw away all contents.

The OraQuick™ HIV Self-Test is an in-vitro diagnostic home-use test for HIV (HIV-1 and HIV-2) in oral fluid. This test works by looking for your body's response (antibodies) to fighting the HIV virus.

IMPORTANT

- Please follow the testing directions carefully to be sure the results are correct.
- You must be 14 or older to use this test.
- A positive result with this test does not mean that you are definitely infected with HIV, but rather that additional testing should be done in a medical setting.
- A negative result with this test does not mean that you are definitely not infected with HIV, particularly when exposure may have been within the previous 3 months.
- If your test is negative and you engage in activities that put you at risk for HIV on a regular basis, you should test regularly.
- This product should not be used to make decisions on behavior that may put you at increased risk for HIV.

Test Performance

- In a clinical study, 4,999 people who were unaware of their HIV status were given the OraQuick™ HIV Self-Test to take home and use. The researchers compared the OraQuick™ HIV Self-Test results with laboratory test results performed by a trained professional. The laboratory based results show that a total of 96 people were HIV positive and 4,903 people were HIV negative. The comparison of results was as follows:
 - 99.9% of people (4,902 out of 4,903) correctly reported a negative test result. The lab and OraQuick™ HIV Self-Test found the same result. This means that 1 out of 4,903 people not infected with HIV reported a positive test result even though that person was really not infected with HIV. This is called a "false positive".
 - 91.7% of HIV positive people (88 out of 96) tested reported a correct test result. This means that 8 out of 96 people who actually were infected with HIV reported a negative test result even though they had HIV. This is called a "false negative".
 - In addition only 1.1% of study subjects (56 out of 5,055) failed to obtain a test result.

Warnings and Precautions

- Most people feel a little bit anxious when taking an HIV test. But, if you feel very anxious about taking the test, you may want to wait until you are calmer to take it, or get tested by your doctor or local clinic.
- Make sure you are taking the test in a place with good lighting.
- If you are on antiretroviral drugs (ARVs), including for HIV treatment, pre-exposure prophylaxis (PrEP), or post-exposure prophylaxis (PEP), you may get a false result.
- If you have participated in a HIV vaccine clinical trial, you may get a positive result using this test, but it may not mean that you are infected with HIV. You should seek follow-up with the research group.
- Do not use if the box seal is opened or any of the contents are opened or missing.
- If today is after the expiration date on the outside of the box, do not use this test.
- Do not open any of the packets until you are ready to begin your test.
- Do not eat, drink or use oral care products (such as mouthwash, toothpaste or whitening strips) 30 minutes before starting the test.
- Remove dental products such as dentures or any other products that cover your gums prior to the oral collection.
- Do not use the test if it has been exposed to household cleaning products.
- Do not use this test if it has been stored outside the acceptable temperature of 36°-80° F (2°-27° C).

Kit contents:

Pouched device(s) with a test stick and test tube (containing 1 mL of fluid), and (one of each of) a test stand, Instructions for Use, and the "HIV, Testing, & Me" booklet.

If any components are missing from the kit or if any of the pouches have been opened, do not use the test. Please contact the OraQuick™ Support Center toll free at 1-866-436-6527.

Materials needed but not provided:

- A timer, watch or something that can time 20 to 40 minutes.
- Eyeglasses – If you use eyeglasses to read, you should wear them while reading your test results.

Questions & Answers

1. **What does the test do?**
The OraQuick™ HIV Self-Test is an in-vitro diagnostic home-use test for HIV (HIV-1 and HIV-2) in oral fluid. This test works by looking for your body's response (antibodies) to fighting the HIV virus. A positive result is preliminary and follow-up confirmatory testing is needed.
2. **What kinds of events put me at risk for HIV?**
 - Sex (vaginal, oral or anal) with multiple sex partners
 - Sex with someone who is HIV positive or whose HIV status you don't know
 - Sex between a man and another man
 - Using illegal injected drugs or steroids
 - Shared needles or syringes
 - Exchanged sex for money
 - Having been diagnosed or treated for hepatitis, tuberculosis or a sexually transmitted disease like syphilis
3. **How soon after a risk event can I test myself?**
This test detects HIV infection if used 3 months after a risk event. If you want to be tested before 3 months, you should go to your doctor or local healthcare provider. If you do not know where to get tested, you can call the OraQuick™ Support Center toll free at 1-866-436-6527 and a Support Center representative can provide you with the name of a healthcare provider or a clinic in your area.
4. **Why shouldn't I use this test right after a risk event?**
When you have been infected with the HIV virus, your body tries to defend against the HIV virus by producing antibodies to it. These antibodies can be found in your blood or oral fluid. It takes your body up to 3 months to produce these antibodies at levels that can be detected by this test.
5. **Can someone help me with directions on how to take the test correctly?**
Yes, the OraQuick™ Support Center is available 8:00AM to 8:00PM ET, 7 days a week toll free at 1-866-436-6527. A Support Center representative will assist you with any questions you may have on performing the test or connecting you with a healthcare provider in your area. Additionally you can go to www.oraquick.com to view a video on how to take the test. All calls are completely confidential.
6. **Can someone help me read my results?**
The OraQuick™ Support Center is available 8:00AM to 8:00PM ET, 7 days a week toll free at 1-866-436-6527. A Support Center representative will assist you with any questions. Since this is a visually read test, they cannot actually read your results; however, they can help you to understand your test result.
7. **How often should someone test for HIV?**
How often you should get tested for HIV depends on your circumstances. If you have never been tested for HIV, you should be tested at least once.

The Centers for Disease Control and Prevention (CDC) recommends being tested at least once a year if you do things that can result in HIV infection. These include:

- Sex (vaginal, oral or anal) with multiple sex partners
- Sex with someone who is HIV positive or whose HIV status you don't know
- Sex between a man and another man
- Using illegal injected drugs or steroids
- Shared needles or syringes
- Exchanged sex for money
- Having been diagnosed or treated for hepatitis, tuberculosis or a sexually transmitted disease like syphilis

If you have been exposed to events that could result in HIV infection, you should test 3 months after the event. If you feel you are at risk of being infected with HIV, it is a good idea to test regularly.

8. **What does a negative result mean?**
A negative result means that the test has not detected HIV antibodies (your body's way of fighting the virus). If it has been at least 3 months since you have had a risk event and you followed the directions carefully, you likely do not have HIV. However, as with any test, there is a chance for false results. For this test 8 out of 96 people who actually were infected with HIV reported a negative test result even though they had HIV. This is called a "false negative". To reduce the chance of false results, be sure to use the test correctly. If it has been less than 3 months since you have had a risk event, wait until the full 3 months have passed to take the test. If you did not follow the directions carefully, you should take the test again to be sure your result is accurate.
9. **What should I do if I get a negative result?**
Please refer to the booklet "HIV, Testing & Me" to learn ways that you can reduce your risks. If you feel you are at risk of being infected with HIV, it is a good idea to test regularly.
10. **Will the results of this test tell me if it is safe to have unprotected sex?**
No, you should not use this test to make decisions on behavior that may put you at increased risk for HIV, such as having unprotected sex.
11. **What is a false negative?**
When a person who is actually infected with HIV receives a negative test result even though they are HIV positive.
12. **What can cause a false negative result?**
A false negative result can occur for the following reasons:
 - If you have had a risk event less than 3 months prior to taking the test
 - Incorrectly reading test result as negative
 - Not following the test directions carefully
 - If you wore dental products such as dentures or any other products that cover your gums while swiping your gums

13. **What does a positive result mean?**
A positive result means that you may have HIV. **A doctor, clinic or healthcare professional must confirm your OraQuick™ HIV Self-Test result.**
14. **What should I do if I get a positive result?**
You need to follow up with a doctor or healthcare provider to get a confirmatory test. At that time your doctor or healthcare provider will discuss the next steps that need to be taken. You should not use a second OraQuick™ HIV Self-Test to confirm a positive result.
If you need help locating a healthcare provider in your area, please call the OraQuick™ Support Center toll free at 1-866-436-6527.
15. **What is a false positive result?**
When a person who is not infected with HIV receives a positive test result even though they are HIV negative.
16. **What can cause a false positive result?**
A false positive result can occur for any of the following reasons:
 - Incorrectly reading test result as positive
 - Not following the test directions carefully
 - Not waiting 30 minutes after eating, drinking, or using oral care products before taking the test
 - If you have participated in a HIV vaccine clinical trial
 - Swiping each gum more than once during oral collection
17. **Is there a way to find out where I can get additional help or care?**
You can call the OraQuick™ Support Center toll free at 1-866-436-6527 and a Support Center representative will be able to provide you with a list of healthcare providers in your area. You may also go to www.oraquick.com.
18. **How can I tell that my test is working correctly?**
If your test is working correctly you will see a complete line next to the "C" on your Test Stick. If there is no line or an incomplete line next to the "C" your test did not work. In this case, the result is considered "invalid" and will need to be repeated with a new test device.
19. **What should I do if my test is not working?**
If you believe your test may not be working correctly, please call the OraQuick™ Support Center toll free at 1-866-436-6527 and a Support Center representative will assist you.
20. **How can I get additional information about HIV/AIDS?**
The OraQuick™ Support Center is available 8:00AM to 8:00PM ET, 7 days a week and can provide you with information about HIV/AIDS. The toll free number is 1-866-436-6527 or you can go to www.oraquick.com.
21. **Do any drugs or medications affect the test?**
To date there is no evidence that the use of antibiotics or medications (not HIV related) affect the test results; however, individuals on treatment for HIV should not use this test.

EXPLANATION OF SYMBOLS

LOT	Batch Code	REF	Catalog Number	Use By	DEVICES	Devices	Date of Manufacturing
Do Not Reuse	DEV SOL VIAL	Developer Solution Vial	Caution, Consult Accompanying Documents	Consult Instructions for Use	EXP	Date of Expiration	
Temperature Limitation	IVD	In Vitro Diagnostic Medical Device	Manufacturer				

OraSure Technologies, Inc.
220 East First Street, Bethlehem, PA 18015 USA
www.OraSure.com

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Orasure Technologies, Inc.